

# Adjustable Nasopharyngeal Airway

*Used to provide a patent airway*

Not Made  
with Natural  
Rubber Latex



Single  
Use



Do Not Use  
if Package  
is Damaged

STERILE R

Sterilized Using  
Gamma Ray



Do Not  
Resterilize

R<sub>X</sub> ONLY

For Professional  
Use Only

## Instructions to provide a patent airway:

1. Lubricate the airway with water soluble lubricant.
2. Gently insert airway into nostril.
3. Do not force. Gently rotate the airway to ease entry.

**SunMed**

[www.Sun-Med.com](http://www.Sun-Med.com)



Manufactured for SunMed  
2710 Northridge Dr. NW Suite A  
Grand Rapids, MI 49544 USA

Made in Malaysia SunMed is a registered trademark in the USA

1-5072 IFU Rev. 1