



Disposable Laryngoscope
Recycling Program

Intended Use: For use only with SunMed's SunOne System disposable laryngoscope blades and handles.

SunCycle Program Components

- 2 (or 3) White SunCycle 7.5 gallon recycling containers with SunCycle label and absorbent pad in bottom
- 4-mil small biohazard-labeled plastic bags for each recycling container
- Packaging tape and tie wraps
- Inner brown shipping box pre-labeled with prepaid return shipping label
- 4-mil large biohazard-labeled plastic bag for lining inner brown shipping box

WARNINGS and CAUTIONS

- ⚠ DO NOT USE for disposable sharps.
- ⚠ DO NOT DISCARD absorbent pads found in the bottom of each recycling container.
- ⚠ PLACE ONLY SunMed disposable fiber optic laryngoscope blades and/or handles in the SunCycle container.
DO NOT PLACE mercury or lead-containing devices, medical waste, disposable sharps or hazardous waste materials into the SunCycle container.
- ⚠ DO NOT OVERFILL, COMPRESS OR FORCE MATERIALS INTO THE SUNCYCLE CONTAINER.
- ⚠ FOLLOW proper packaging and return shipping steps included in these instructions.
- ⚠ ONLY USE SunCycle-labeled and *TakeAway Recycle System* components to package and ship container.

NOTE: Maximum return container weights are 35lbs for two SunCycle containers and 50lbs for three SunCycle containers. See steps below for proper use and return shipping instructions.

Preparing SunCycle Container(s) for Use

1. Open the exterior shipping box. Inside is a second brown box – REMOVE this inner box from the shipping box and SAVE. This inner box is pre-labeled for return shipping once the SunCycle containers are full.
2. Remove the white SunCycle containers and place in appropriate locations for disposable laryngoscope collection.

Your SunCycle recycling container is now ready for use.

Using SunCycle Container(s)

1. Place the SunCycle container(s) near the point of use. Keep the container(s) away from children and untrained persons.
2. Following use of any SunMed disposable fiber optic laryngoscope blade and/or handle, dispose device(s) in the SunCycle container.
3. TO PREVENT INJURY, DO NOT OVERFILL THE SUNCYCLE CONTAINER.
4. Once the SunCycle container is full, firmly secure and lock the lid.
5. Follow instructions for proper packaging and return shipping as directed.

Proper Packaging and Return Shipping of SunCycle Container(s)

IMPORTANT! Only use SunCycle-labeled and TakeAway Recycle System components to package and ship containers

1. Place each filled plastic container inside one of small 4-mil plastic bags and securely close using tie wraps provided (**Figure 1.**)
2. Place each bagged SunCycle container inside the large 4-mil plastic bag box liner located in the prepaid inner box (**Figure 2.**)
3. Close large plastic liner around all bagged SunCycle containers and secure with tie wraps provided (**Figure 3.**)
4. Close flaps on the return shipping box in order as indicated and securely close using tape provided (**Figure 4.**) Make sure UPS prepaid return shipping label is visible.
5. Give to a UPS driver or schedule a UPS pick up by calling 800-772-5657. The return freight is prepaid; no additional freight charges are required.

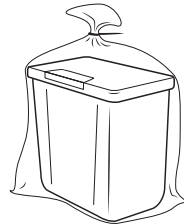


Figure 1. Securely seal SunCycle container with small plastic bag and tie wraps.

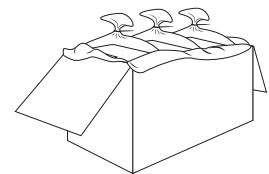


Figure 2. Line the return shipping box with large plastic bag. Place each bagged SunCycle container in the shipping box.

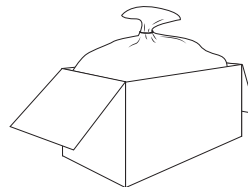


Figure 3. Close and secure the liner bag around the recycling containers using the tie wrap.

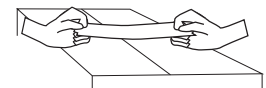


Figure 4. Close flaps on return shipping box in order indicated and securely close using tape provided.