

## Transport Circuit with 19mm Adjustable PEEP (5-20 cm H<sub>2</sub>O) and 9' Expandable Corrugated Tubing

### INDICATIONS

*Intended to provide positive end expiratory pressure to patients during exhalation while attached to a ventilator. The PEEP valve is adjustable from 5-20 cm of H<sub>2</sub>O.*



### CAUTIONS and WARNINGS

- All components of this device are MRI compatible
- This device is to be used only by personnel properly trained in the use and possible hazards and complications associated with this device
- All users of this device should read instructions for use and assembly of the circuit and associated transport equipment
- When attaching any item after the patient connector, dead space will be added to the circuit; the user must evaluate the amount and side effects caused by adding this dead space
- Actual PEEP may vary with patient lung compliance and resistance; always verify PEEP with a manometer
- If PEEP is not desired, set the PEEP at minimum setting or remove it
- Constantly monitor patient for effectiveness of ventilation
- This is a single use device; DO NOT sterilize with steam autoclave, ethylene oxide gas or chemical disinfectant

### DIRECTIONS FOR USE

1. Connect the patient hose to the ventilator per equipment protocol.
2. Prior to patient connection, attach tubing to manifold. Check that all connections are secure. Extend tubing as required. Pressurize to confirm there is no leakage or occlusion of circuit.
3. The PEEP Valve is attached to the patient exhalation port; select desired PEEP setting by:
  - Tightening cap to increase pressure
  - Loosening cap to decrease pressureActual PEEP may vary with patient lung compliance and resistance; always verify PEEP with a manometer. If PEEP is not desired, set the PEEP at minimum setting or remove it.
4. Attach the patient connector to the airway interface (endotracheal tube or mask).
5. Review the ventilator settings and turn on the ventilator; perform a pre-use check as described in ventilator user manual.

### COMPATIBILITY

This device is compatible with Smith Pneupac® ParaPAC® Ventilators.

### PEEP Valve Range

5 to 20 cm H<sub>2</sub>O  
MIN: >4 cm H<sub>2</sub>O  
MAX: <20 cm H<sub>2</sub>O  
DEAD SPACE: <20 ml



Manufactured for Ventlab, LLC, a Subsidiary of SunMed  
2710 Northridge Dr. NW, Suite A | Grand Rapids, MI 49544 USA  
[www.Sun-Med.com](http://www.Sun-Med.com)