



19mm MRI Compatible PEEP Valve

5-20cm H₂O

REF VP705 / 1 EACH



Not Made
with Natural
Rubber Latex

Warnings

- This device is to be used only by personnel trained in CPR procedures.
- Actual PEEP may vary with patient lung compliance and resistance.
- Always verify PEEP with manometer.
- Constantly monitor patient for effectiveness of ventilation.
- DO NOT sterilize with steam autoclave, ethylene oxide gas or chemical disinfectant.

Preparation for Use

1. Remove PEEP valve from package.
2. Connect PEEP valve to exhalation port of manual resuscitation bag. Squeeze the bag to insure that PEEP valve does not interfere with compression of bag.

Directions for Use

Select desired PEEP setting by:

- *Tightening* cap to *increase* pressure.
- *Loosening* cap to *decrease* pressure.



Manufactured for Ventlab, LLC, a Subsidiary of SunMed
2710 Northridge Dr. NW, Suite A | Grand Rapids, MI 49544 USA
www.Sun-Med.com

Made in China by SunMed AP, LTD with US material;
inspected and packaged in USA.

VP705.IFU Rev1